

## HOW TO ASSEMBLE YOUR



### Toilet Seat Riser with Arms

Item # 8343-R, 8344-R  
Capacity not to exceed 300 lb



**No tools required**  
**All parts included**


**Easy to follow**  
**step by step**  
**instructions**



Seat Riser



Arms (left and right)

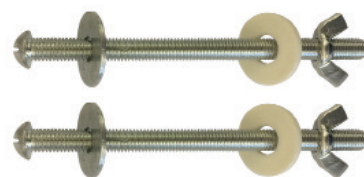

Mounting Screws, Metal Washers,  
Plastic Washers and Wing Nuts

- 1** Remove all parts from box:  
(1) **Seat Riser**, (2) **arms (left and right)**, (2) **mounting screws**, (2) **metal washers**, (2) **plastic washers** and (2) **wing nuts**
- 2** To attach the **arms** to the **Seat Riser**, align each **arm** with the **Seat Riser** tubing. The longer side of the **arm** should be facing the front of the **Seat Riser**. Press the brass buttons on the **arm** tubes and insert the arm tubing into the **Seat Riser** tubing. You should hear a click when the brass buttons are through the first tubing holes (the second tubing hole on the **Seat Riser** is for assembly purposes only).
- 3** Remove the existing toilet seat and lid from the toilet frame by unscrewing seat screws located at the rear of the toilet seat.
- 4** Place the **Seat Riser** on the toilet frame. The mounting holes on the rear of the **Seat Riser** should line up with the mounting holes on the rear of the toilet frame.
- 5** Place toilet seat on **Seat Riser** and align mounting holes with toilet frame. Place the **metal washers** onto the **mounting screws**, and place into mounting holes of toilet seat, the **Seat Riser**, and toilet frame. Then place the **plastic washers** and the **wing nuts** onto the **mounting screws**. Do not tighten the **wing nuts** completely.
- 6** Be sure the toilet seat is straight and sits flush on the **Seat Riser**, then tighten the **wing nuts**.

#### IMPORTANT WARNING!

**READ THESE DIRECTIONS CAREFULLY BEFORE USE**  
**FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY**

- Be sure **Seat Riser** is in place and secure before each use
- Do not stand on **Seat Riser**
- Arms must be secure in **Seat Riser**

Questions?

Please contact us at

**1-800-557-6682**

**info@novamedicalproducts.com**

**novamedicalproducts.com**

Thank you for choosing NOVA!

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

#### Connect with us!

Facebook: novamedicalprod

Twitter: novamedicalprod

Twitter: novawalkertalk

Pinterest: novamedicalprod

## WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

.....  
Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Please regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.  
.....