

HOW TO INSTALL YOUR



Wall Grab Bar

Models: 8012CH-R, 8016CH-R, 8018CH-R, 8024CH-R, 8032CH-R, 8012PC-R, 8016PC-R, 8018PC-R, 8024PC-R, 8032PC-R

Capacity not to exceed 250 lb

Easy to follow
step by step
instructions



TOOLS NEEDED



Electric Drill



Pencil



Drill Bit Set



Phillips Screwdriver



Stud Finder

WHAT'S IN THE BOX

- a. (1) Grab Bar (chrome or ivory) c. (6) Plastic Anchors (for tile only)
b. (6) Mounting Screws d. (2) Rubber Seals



a.



b.



c.



d.

- 1** Locate stud behind existing shower wall to determine where the grab bar can be safely installed. Clean portion of wall to be used for mounting

⚠ WARNING: BOTH GRAB BAR FLANGES MUST BE MOUNTED DIRECTLY INTO WALL STUDS OR USE PLASTIC ANCHORS WHEN NO STUDS ARE AVAILABLE.

- 2** Position mounting flanges and use a pencil to mark where holes need to be drilled. Use a 1/4" drill bit to drill pilot holes through shower wall into wood or metal stud.

NOTE: If wall is tiled, drill through tile ONLY with a 5/16" drill bit, and then switch to 1/4" drill bit to complete the hole.

- 3** Place rubber seals and mounting flanges against the wall, and align with holes (**Figure 1**). Insert provided mounting screws and screw until tightened.

NOTE: If wall is tiled, insert plastic anchors into holes before mounting.

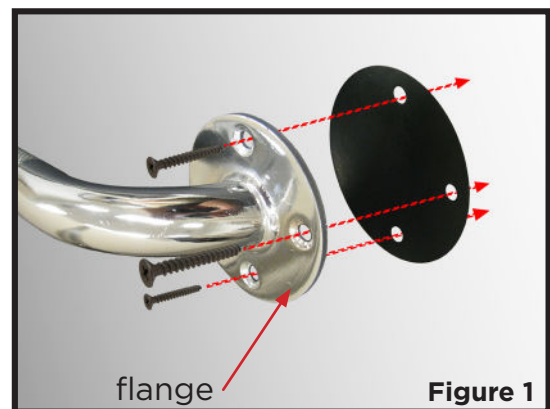


Figure 1

⚠ WARNING

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage to tile or fiberglass walls or injury resulting from improper installations or misuse of this product.**

HOW TO USE

Grasp grab bar when entering and exiting bathtub or shower. Use to stabilize when needed.

CARE AND MAINTENANCE

Check regularly to make sure the Wall Grab Bar is securely mounted to the wall. Use a mild detergent and warm water to clean.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame - Limited Lifetime Warranty

Non-Metal/Metal parts - One Year Limited Warranty
(rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and its parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Use **EXTREME** caution on wet tub or floor surfaces
- **DO NOT** stand or sit on grab bar
- **DO NOT** use grab bar as a step or platform to reach above tub
- **DO NOT** use if grab bar is damaged. Check frequently for damage or looseness.
- **DO NOT** put entire weight on grab bar as it is rated for persons weighing up to **250 lb** when properly installed

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!

novamedicalproducts.com

facebook: [novamedicalprod](#)

twitter: [novamedicalprod](#)

twitter: [novawalkerstalk](#)

YouTube: [NOVA Medical Products](#)